Document No	Issue Date	Revision No	Revision Date
<i>No documento</i>	<i>Data di rilascio</i>	<i>No revisione</i>	<i>Data revisione</i>
TF05-IFU05	01.04.2019	02	05.11.2021

Perfluorodecaline
Perfluorodecalin

Company Name <i>Nome dell'azienda</i>	:	MERAN TIP ÜRÜNLERİ TEKNOLOJİ SAN. TİC. LTD. ŞTİ.
Authorized Representative <i>Rappresentante Autorizzato</i>	:	Cemil BAYTAŞ / General Manager
Address <i>Indirizzo</i>	:	Ferhatpaşa Mah Arif Nihat Sok. A Blok No:4/1A Çatalca/İstanbul / TÜRKİYE
Phone <i>Telefono</i>	:	+90 216 557 87 55
Web <i>Web</i>	:	www.merantip.com

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Model and size information of the Perfluorodecalin products produced by us are shown in **TF05.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:**Destinazione d'uso:**

Perfluorodecalin (C10F18) is a fluorocarbon. Perfluorocarbon liquids are established tools for intraoperative hydrokinetic retinal manipulation. Perfluorodecalin is used as per-operative devices, for transitory intra-ocular endotamponade. In particular retinal detachment (RD), RD with giant tear, RD with proliferative vitreal retinopathy (PVR) including diabetic proliferative retinopathy, retrieval of luxated natural or artificial lens in the vitreous.

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Indications:

Indicazioni:

Perfluorodecalin is used for generally retinal detachment (RD), RD with giant tear, RD with proliferative vitreal retinopathy (PVR) including diabetic proliferative retinopathy, retrieval of luxated natural or artificial lens in the vitreous.

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Patient Population:

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There is no restriction on patient population. The appropriate patient selection is the responsibility of the surgeon.

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Contraindications:

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The product should not be allowed to infiltrate under the sub-retinal tissues since there are no specific studies on tolerance.

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Complications:
Complicazioni:

If the product is not completely removed after intraocular surgery, retinal damage and toxicity can occur due to its high specific gravity.

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Side Effects:
Effetti collaterali:

There are no known side effects of the product.

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The products should be used by experienced surgeons who are experts in the retinal surgical technique.

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

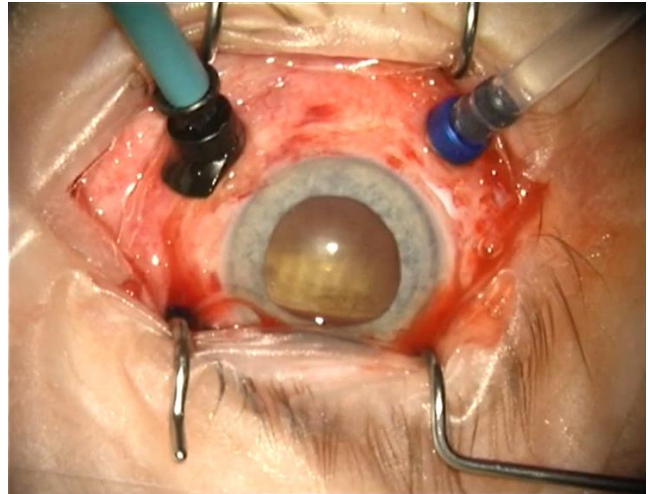
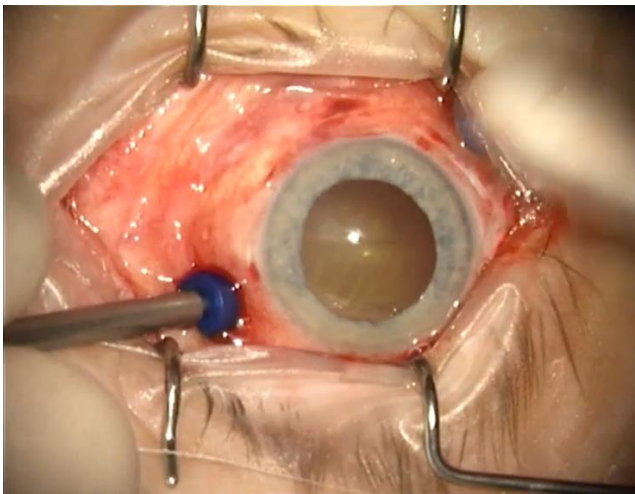
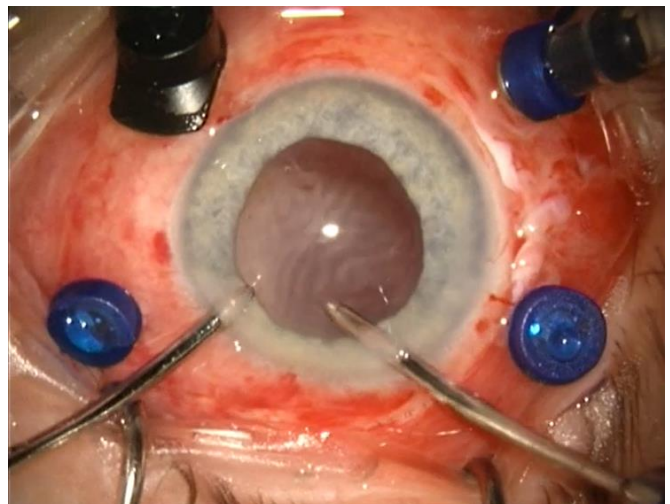
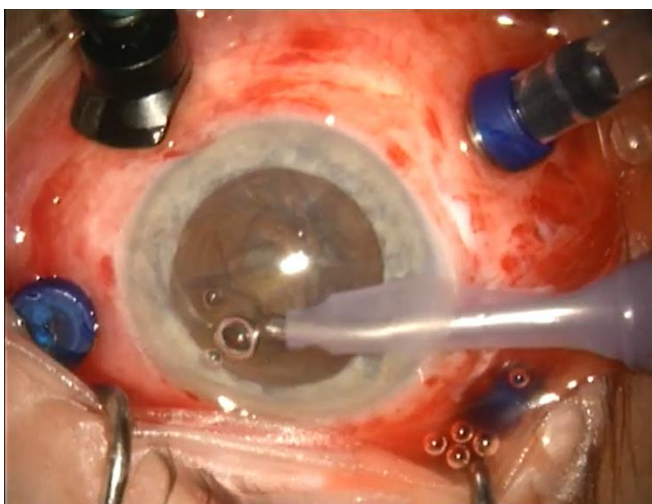
- A cannula is inserted into the trocar placed in the patient's eye. Perfluorodecalin is then started to be injected into the subretinal area of the patient's eye. The patient's eye is illuminated with the help of Chandelier Light and the operation area is easily seen by the surgeon.

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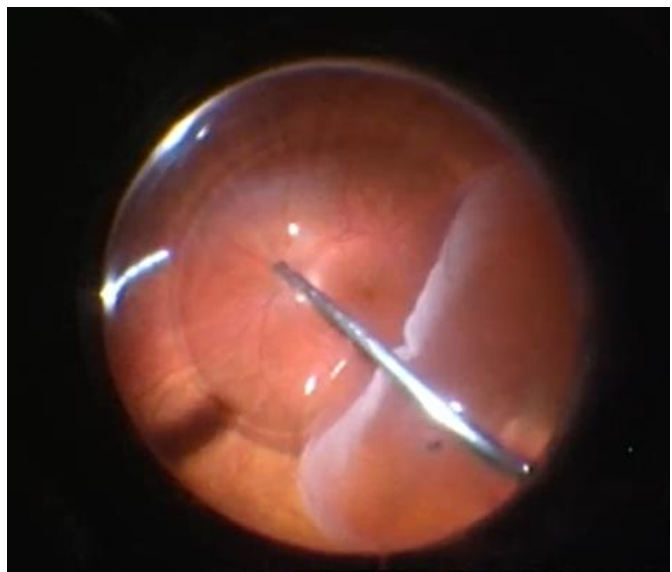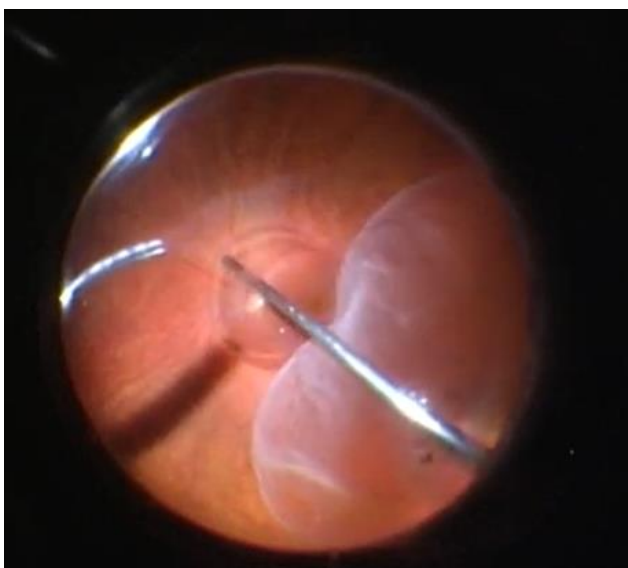- Blood, residues etc. in the sub-retina area with retinal detachment are cleaned. Perfluorodecalin is then continued to be injected into this retinal detachment area. While injecting perfluorodecalin, residues, blood, fluid, etc. are removed from the retinal detachment area.

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

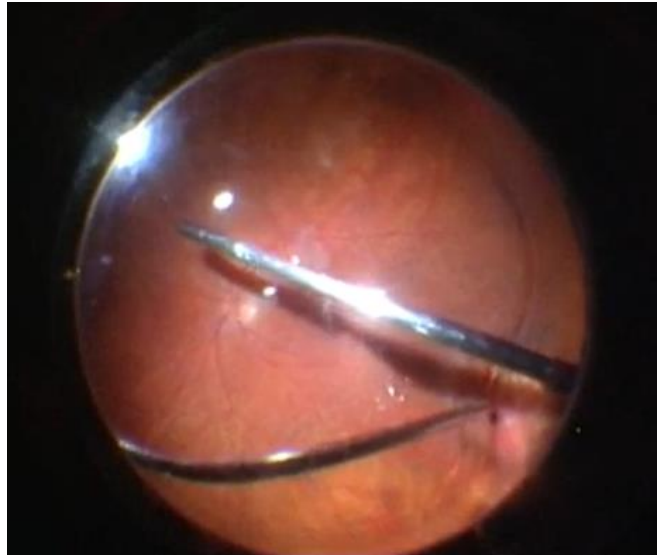
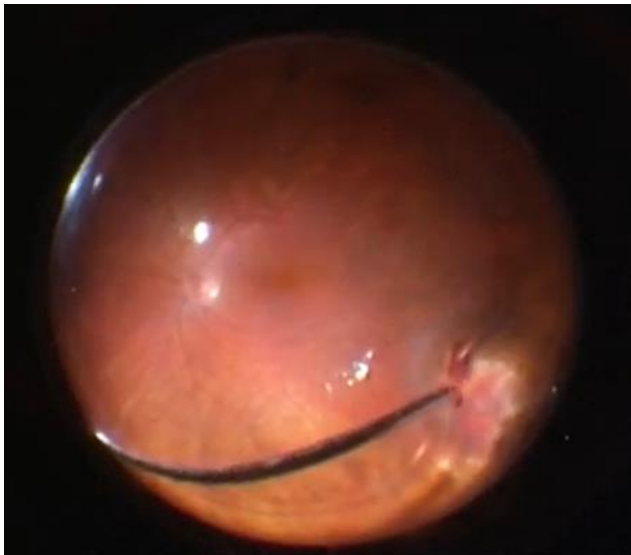
Revision No
No revisione
02

Revision Date
Data revisione
05.11.2021

- Then, according to the type of the disease, the appropriate treatment for the retina is applied. In this operation, laser treatment was applied to the retinal detachment area of the patient. The Intra Ocular Lens is then placed in the patient's eye.

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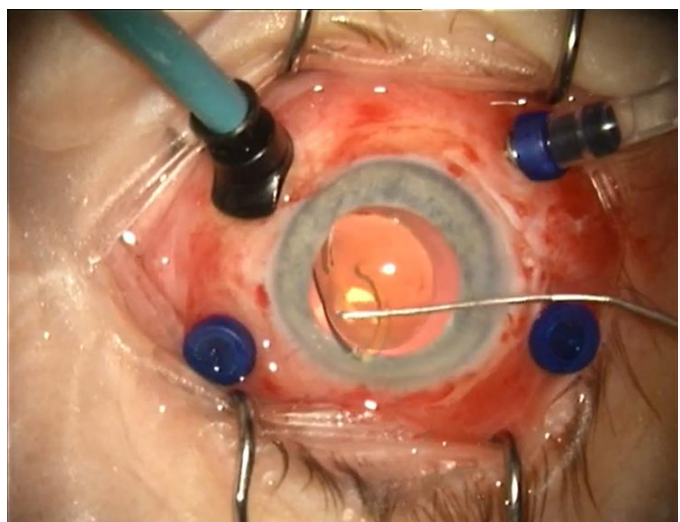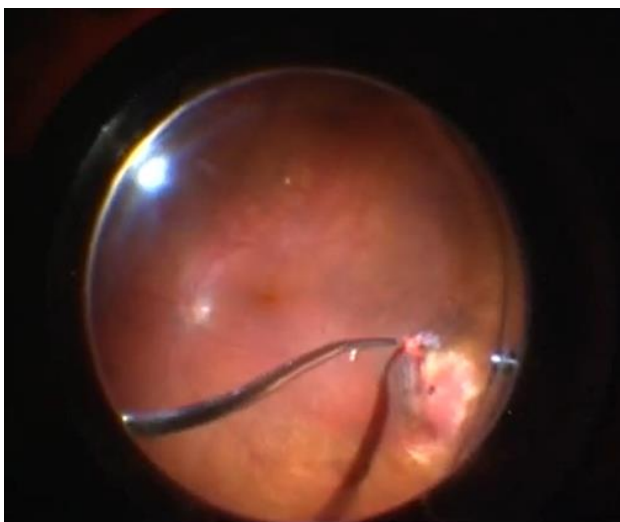
- After the procedure is completed, Perfluorodecalin should be removed from eye. Perfluorodecalin is removed with one of the cannulas placed in the trocars, while air is introduced into the eye from the other. In this way, the exchange of perfluorodecalin and air takes place. Then, the perfluorodecalin residues remaining in the patient's retina are also cleaned.

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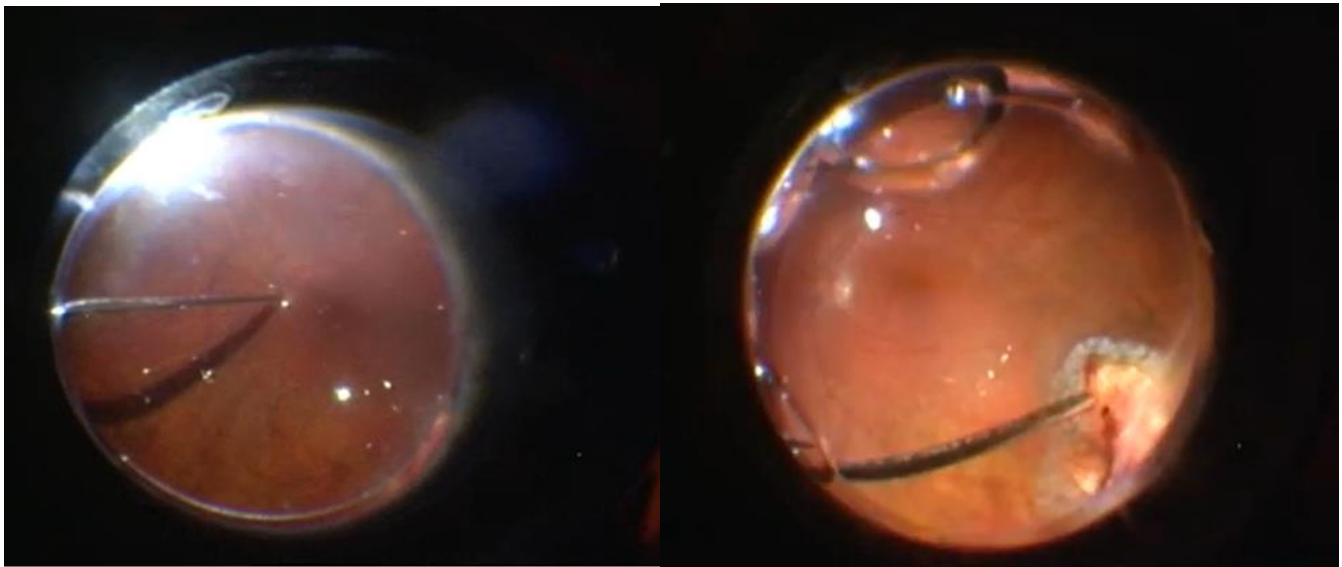The products are sterilized with Steam Sterilizer and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Shelf life of the product is determined as 3 years as a result of stability study.

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Contact Duration of the Product with Body:

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Perfluorodecalin is in contact with the patient's eye tissue, especially the retina, throughout the operation period (<24 h). After the operation is completed, the product is removed from the patient's eye.

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Product Storage and Shipping Conditions

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Protect the product against direct sunlight during shipment and storage.

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Keep the product dry in any environment.

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Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Medical Equipment Used with Products:

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The syringe and / or cannula used for intraocular injection of Perfluorodecalin.

Document No	Issue Date	Revision No	Revision Date
<i>No documento</i>	<i>Data di rilascio</i>	<i>No revisione</i>	<i>Data revisione</i>
TF05-IFU05	01.04.2019	02	05.11.2021

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Accessory:

Accessori:

The single use needle and syringe is presented to the users with vial product as an accessory.

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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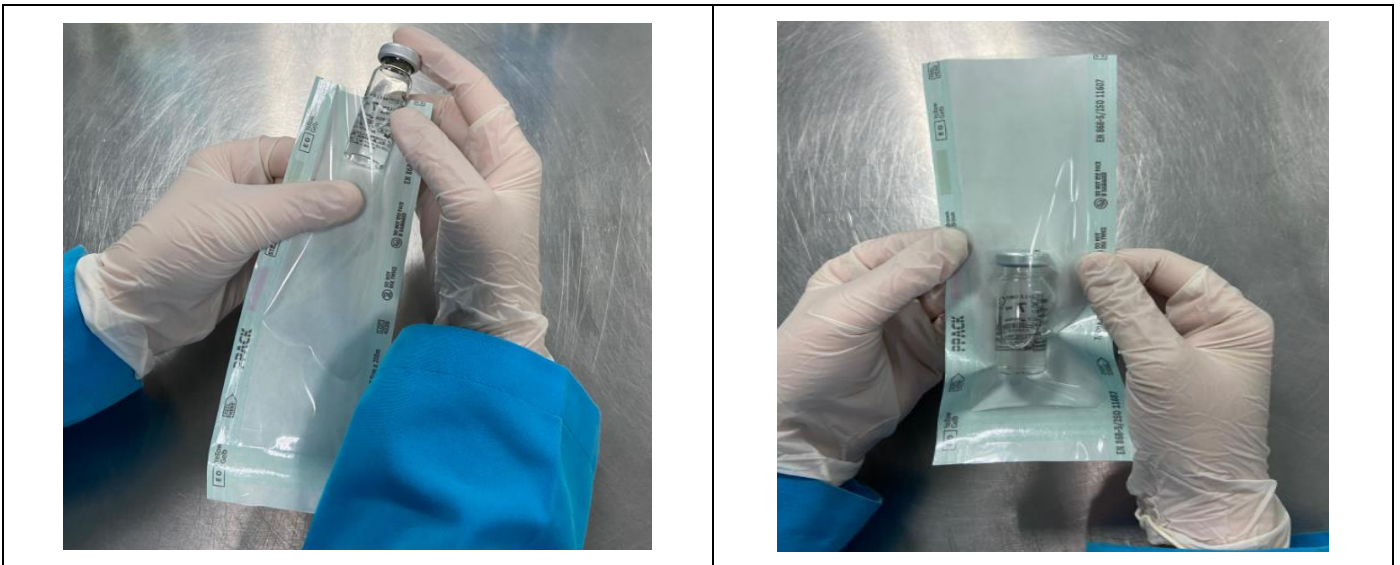
Revision No
No revisione
02

Revision Date
Data revisione
05.11.2021

The packaging sample of our products is as seen in the photos below.

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Revision No
No revisione
02

Revision Date
Data revisione
05.11.2021

Symbols Prepared According to EN 15223-1 Standard and Their Meanings

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