User Manual

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IRRIGATION, ASPIRATION AND POLISHER CANNULA *IRRIGATION, ASPIRATION AND POLISHER KANÜL*

Company Name	:	MERAN TIP ÜRÜNLERİ TEKNOLOJİ SAN.TİC.LTD.ŞTİ
<i>Şirket Adı</i>	:	
Company Representative	:	Cemil BAYTAŞ / General Manager
<i>Firma temsilcisi</i>	:	
Address	:	Ferhatpaşa Mah. Arif Nihat Sok. A Blok No:4/1A
<i>Adres</i>	:	ÇATALCA/ İSTANBUL/TÜRKİYE
Phone	:	+90 216 557 8 755
<i>Telefon</i>	:	
Web	:	www.merantip.com
<i>Web</i>	:	

Product Structure and General Features:

The Irrigation, Aspiration and Polisher Cannula is designed for the purposes of irrigation, aspiration and polishing procedures. Cannula tip is made of 304 Stainless Steel. The connection area at the back of the cannula is made of polycarbonate raw material. The products are available for single use and sterile. The cannulas to be used may differ depending on the operation area and the surgeon's choice. Models are available in various diameters and sizes in order to be suitable for every eye structure.

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Product Models:

Ürün Modeli:

Model and size information of the Irrigation, Aspiration and Polisher Cannula products manufactured by us are shown in **TF03.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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Irrigation, aspiration and polishing procedures can be performed with Irrigation, Aspiration and Polisher Cannulas. These cannulas are disposable and used usually in cataract surgeries. Irrigation, Aspiration and Polisher Cannulas used for removing debris from the posterior, anterior, and equatorial regions of the capsular bag. Micro etched tips are slightly roughened for gentle polishing.

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Irrigation, Aspiration and Polisher Cannulas are used in ophthalmic surgeries for irrigation of fluid, aspiration of fluids from eye and polishing procedure of capsule of eye.

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Patients with various eye conditions, especially cataracts.

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It should not be used other than ophthalmic surgery.

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It should not be used on infected eyes.

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If the tip of the Cannula is not sharp, it may damage the tissue of the eye.

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After operation mild chemosis and redness can occur.

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Intraoperative serous occurrence.

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Side Effects:

Yan Etkiler:

There are no known side effects of the product.

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Product Users:

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The product should be used by specialist physicians who have completed the necessary medical training. Users must have operational experience.

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Instruction for Use:

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Irrigation, Aspiration and Polisher Cannulas are restricted for use by surgeons trained in ophthalmic surgery.

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➤ The Cannulas should be opened from the marked are without disruption of the sterility.

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➤ Cannula should be connected to the Equipment by a sterile person according to usage of the product.

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➤ Depending on the use, air, fluids or viscoelastic substances are injected into eye or removed from eye.

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➤ Dispose of used products in accordance with established hospital protocol for hazardous waste.

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Revision Date

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02.06.2023

Factors That Can Affect the Success of The Operation:

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Sterility/Sterilization Method:

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The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Product Storage and Shipping Conditions

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Protect the product against direct sunlight during shipment and storage.

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Keep the product dry in any environment.

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Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Revision Date

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02.06.2023

Medical Equipment Used with Products:

Medical Equipment Used with Products:

The products are used with injectors during surgery.

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Accessory:

Aksesuar:

The product does not have any accessories.

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Shelf Life:

Raf Ömrü:

In line with the stability studies carried out in independent laboratories, the product shelf life was determined as 4 years.

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Packaging, Labeling and Handling:

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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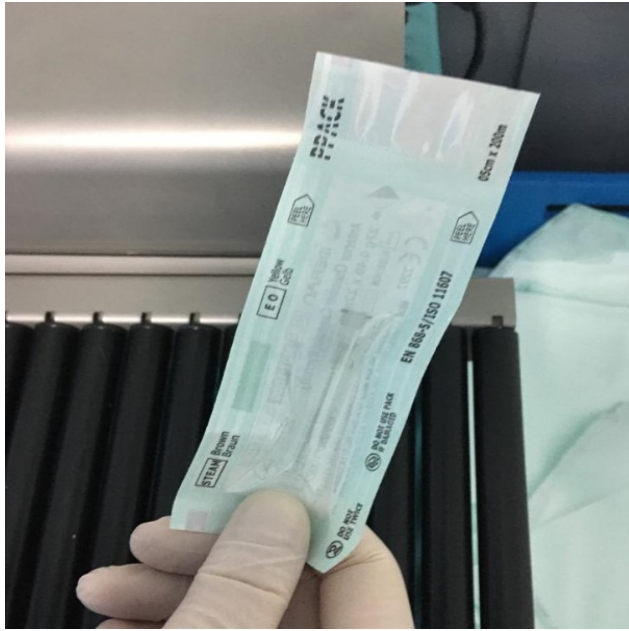
Revision Date

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02.06.2023

The packaging sample of our products is as seen in the photos below.

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Revision Date

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02.06.2023

Symbols Prepared According to EN 15223-1 Standard and Their Meanings

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