## User Manual

**Document No**  
**Doküman No**  
TF07-IFU07-07

**Issue Date**  
**Yayın Tarihi**  
23.09.2020

**Revision No**  
**Revizyon No**  
02

**Revision Date**  
**Revizyon Tarihi**  
01.06.2023

### SURGICAL TABLE DRAPE AMELİYAT MASA ÖRTÜSÜ

**Company Name**  
**Firma Adı** : MERAN TIP ÜRÜNLERİ TEKNOLOJİ SAN. TİC. LTD. ŞTİ.  
**Company Representative**  
**Firma Yetkilisi** : Cemil BAYTAŞ / General Manager  
**Address**  
**Adres** : Ferhatpaşa Mah Arif Nihat Sok. A Blok No:4/1A  
Çatalca/İstanbul / TÜRKİYE  
**Phone**  
**Telefon** : +90 216 557 87 55  
**Web**  
**Web** : [www.merantip.com](http://www.merantip.com)

#### **Product Models:**

##### **Ürün Modelleri:**

Model and size information of the Surgical Table Drape products produced by us are shown in **TF07.TB.01 Annex-1 Product Models Table** with pictures.

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#### **Intended Use:**

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Surgical Table Drape is used in all surgical operations to protect hygiene, to ensure safety against infection risk that may arise from instrument tables to surgical instruments for patients.

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#### **Indications:**

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Surgical Table Drape is used in all surgical operations in order to prevent patients from infection which may arise from instrument tables where surgical instruments are put.

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#### **Patient Population:**

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Patients who undergo surgical operations.

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**Revision Date**  
**Revizyon Tarihi**  
01.06.2023

### **Contraindications:**

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Apart from preventing infection, our products are used for radiation etc. It should not be used to protect against such substances.

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### **Complications:**

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There is no complication of the product.

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### **Side Effects:**

#### **Yan Etkiler:**

There are no known side effects of the product.

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### **Product Users:**

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The product should be used by medical personnel who attend surgical operations.

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### **Usage Information:**

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➤ Before using Surgical Table Drape, check the package to make sure there are no torn holes.

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➤ Take the product into the sterile area and open it from top or bottom of the bag.

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➤ Lay the cover on the instrument table by aligning the barrier at the center on the table.

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➤ At the end of the surgical procedure, collect the cover, place them in a medical waste bag and take them to the medical waste area for disposal.

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### **Sterility/Sterilization Method:**

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The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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### **Shelf Life:**

#### **Raf Ömrü:**

Shelf life of the product is determined as 4 years as a result of stability study.

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### **Product Storage and Shipping Conditions**

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The ambient temperature where the products are stored must be in the range of +5 / +35 ° C and the humidity at maximum 70% rh.

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The products should be stored in a way not to be exposed to direct sunlight and high above the ground to prevent contact with water.

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Revision No  
*Revizyon No*  
02

Revision Date  
*Revizyon Tarihi*  
01.06.2023

### **Medical Equipment Used with Products:**

#### **Medical Equipment Used with Products:**

The products are used with no additional medical equipment.

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### **Accessory:**

#### **Aksesuar:**

The product does not have any accessories.

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### **Packaging, Labeling and Handling:**

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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Packaging steps are demonstrated as following:

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