IRRIGATION AND AIR INJECTION CANNULA
IRRIGATION AND AIR INJECTION KANÜL

Company Name <i>Şirket Adı</i>	:	MMT MERAN MEDİKAL TEKNOLOJİ KOLL. ŞTİ. Ahmet Cemil Baytaş ve Fatma Baytaş
Company Representative <i>Firma temsilcisi</i>	:	Cemil BAYTAŞ / General Manager
Address <i>Adres</i>	:	İstanbul Trakya Serbest Bölgesi Ferhatpaşa S.B. Mah. Ali Rıza Efendi Cad. No:22/201 Çatalca/İSTANBUL/TÜRKİYE
Phone <i>Telefon</i>	:	+90 212 786 6253
Web <i>Web</i>	:	www.merantip.com

Product Structure and General Features:

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The Irrigation and Air Injection Cannula is designed for the use in cataract surgeries and their purpose is to inject various liquids and air on the eye and anterior chamber. Cannula tip is made of 304 Stainless Steel. The connection area at the back of the cannula is made of polycarbonate raw material. The products are available for single use and sterile. The cannulas to be used may differ depending on the operation area and the surgeon's choice. Models are available in various diameters and sizes in order to be suitable for every eye structure.

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Product Models:

Ürün Modeli:

Model and size information of the Irrigation and Air Injection Cannula products manufactured by us are shown in **TF03.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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Irrigation and Air Injection cannulas are disposable and usually used in cataract surgeries and their purpose is to inject various liquids and air on the eye and anterior chamber. It may also be used for removal of air, fluids & viscoelastic substances during an ophthalmic surgery.

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Indications:

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Irrigation and Air Injection cannulas are used to inject air, fluids and viscoelastic substances into eyes during ophthalmic surgeries.

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Patient Population:

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Patients with various eye conditions, especially cataracts.

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Contraindications:

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It should not be used other than ophthalmic surgery.

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It should not be used on infected eyes.

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Complications:

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If the tip of the Cannula is not sharp, it may damage the tissue of the eye.

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After operation mild chemosis and redness can occur.

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Intraoperative serous occurrence.

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There are no known side effects of the product.

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Product Users:

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The product should be used by specialist physicians who have completed the necessary medical training. Users must have operational experience.

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Instruction for Use:

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Irrigation and Air Injection Cannulas are restricted for use by surgeons trained in ophthalmic surgery.

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➤ The Cannulas should be opened from the marked are without disruption of the sterility.

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➤ Cannula should be connected to the Equipment by a sterile person according to usage of the product.

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➤ Depending on the use, air, fluids or viscoelastic substances are injected into eye or removed from eye.

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➤ Dispose of used products in accordance with established hospital protocol for hazardous waste.

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Factors That Can Affect the Success of The Operation:

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Sterility/Sterilization Method:

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The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Product Storage and Shipping Conditions

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Protect the product against direct sunlight during shipment and storage.

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Keep the product dry in any environment.

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Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Medical Equipment Used with Products:

Medical Equipment Used with Products:

The products are used with injectors during surgery.

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Accessory:

Aksesuar:

The product does not have any accessories.

The packaging sample of our products is as seen in the photos below.

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