User Manual

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Document No
Doküman No
TF02-MMT-IFU02

Issue Date
Yayın Tarihi
01.04.2019

Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

Silicone Oil
Silikon Yağı

Company Name
Firma Adı

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Company Representative
Firma Temsilcisi

: Cemil BAYTAŞ / General Manager

Address
Adres

: İstanbul Trakya Serbest Bölgesi Ferhatpaşa S.B. Mah. Ali Rıza Efendi Cad. No:22/201 Çatalca/İSTANBUL/TÜRKİYE

Phone
Telefon

: 0212 786 6253

Web
Web

: www.merantip.com

Product Models:

Ürün Modelleri:

Model and size information of the Silicone Oil products produced by us are shown in **TF02.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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Silicone oil is used as long-term intraocular tamponade in vitreoretinal surgery.

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It is used in severe retinal surgeries such as retinal detachment (RD), retinal tears, proliferative vitreoretinopathy (PVR), traumatic retinal detachment, retinal detachment more complicated by diabetic retinopathy.

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Our products are presented to users in two different forms, in Vial and Syringe.

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Indications:

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Indications for silicone oil may include: retinal detachment (RD), retinal tears, proliferative vitreoretinopathy (PVR), traumatic retinal detachment, retinal detachment more complicated by diabetic retinopathy.

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Patient Population:

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There are no restrictions on the patient population. The choice of appropriate patient is the responsibility of the surgeon. However, the products are generally used in patients who will undergo retinal detachment surgery.

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Contraindications:

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It should not be used in pseudophakic patients with hypersensitive silicon intraocular lenses.

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Revision No
Revizyon No
 04

Revision Date
Revizyon Tarihi
 20.11.2023

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It may cause eye pressure (Glaucoma) in long-term use.

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Retinal detachment may recur after silicone oil is removed.

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Side Effects:**Yan Etkiler:**

There are no known side effects of the product.

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The products should be used by experienced surgeons who are experienced in the surgical technique.

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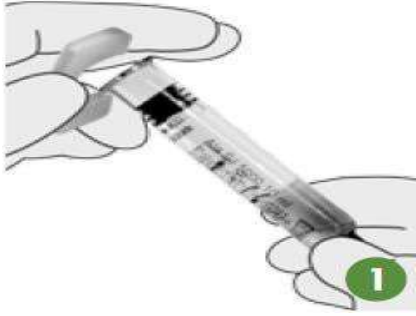
Cerrahi Teknik:

1) Silicone Oil Injection

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There are three methods for silicone oil injection.

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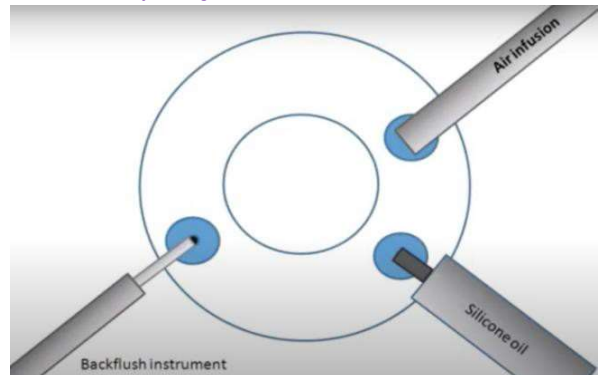
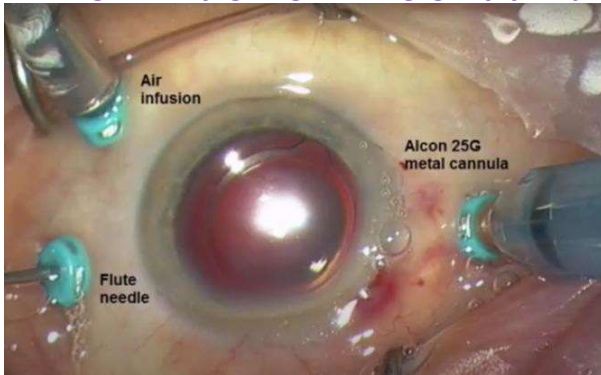
a. **No view to fundus**

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- Various ophthalmic instruments are placed in the patient's eye as shown below.

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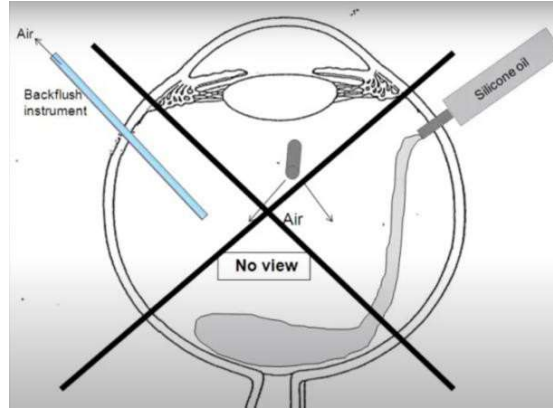
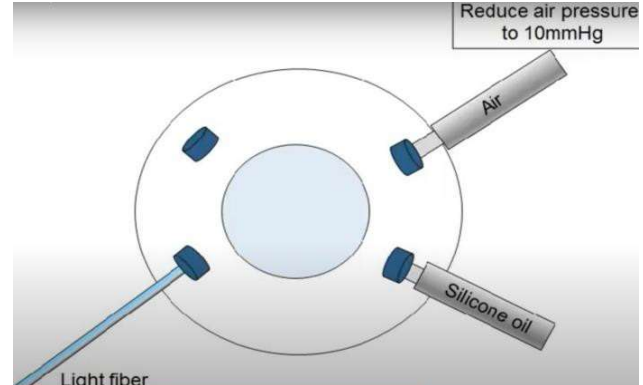
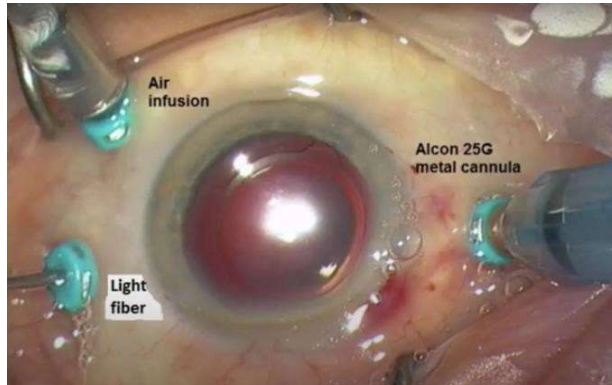
- The extrusion cannula (Flute Needle) has been replaced by a light fiber.

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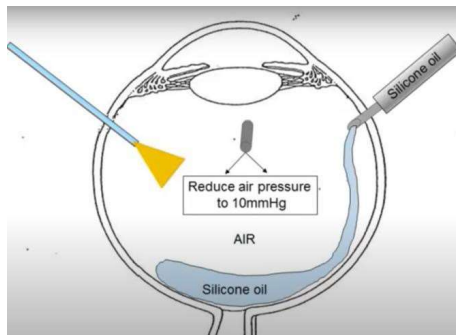
- Reduce the air pressure to 10 mmHG.

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- Fundus images can obtain in this method.

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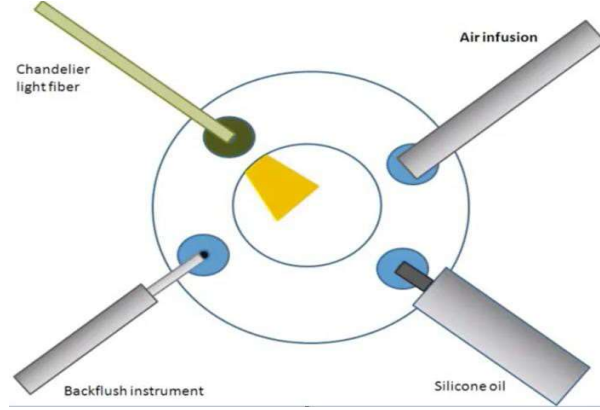
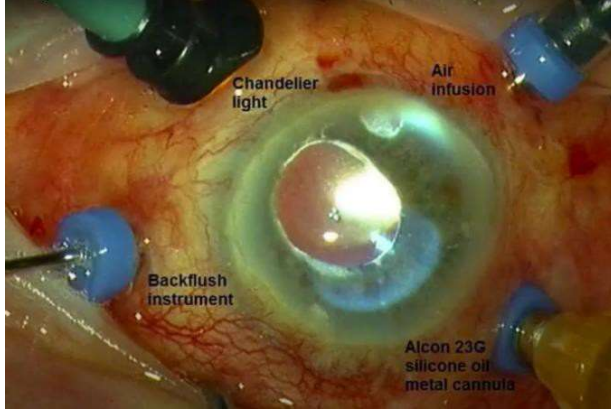
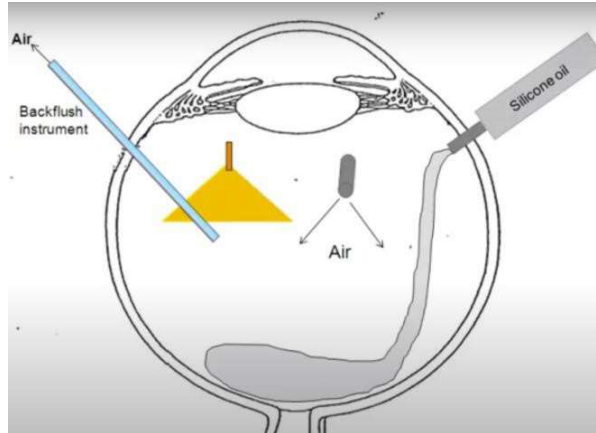
- The situation in the posterior cavity is as follows.

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The application of Chandelier Light method from these methods is as follows.

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- Silicone oil injection port, Air Infusion port, Backflush Instrument and Chandelier Light are placed in the patient's eye by the surgeon. Then, silicone oil is injected into the air-filled vitreous cavity.

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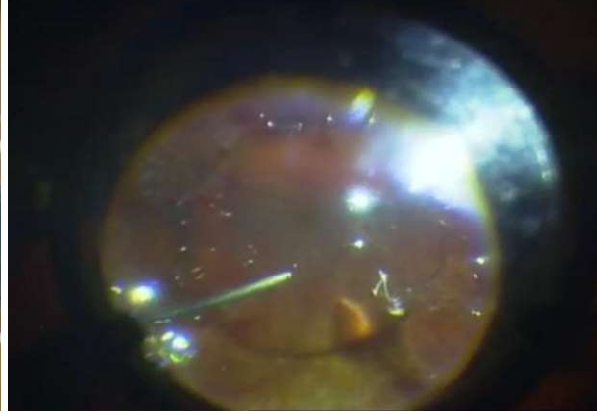
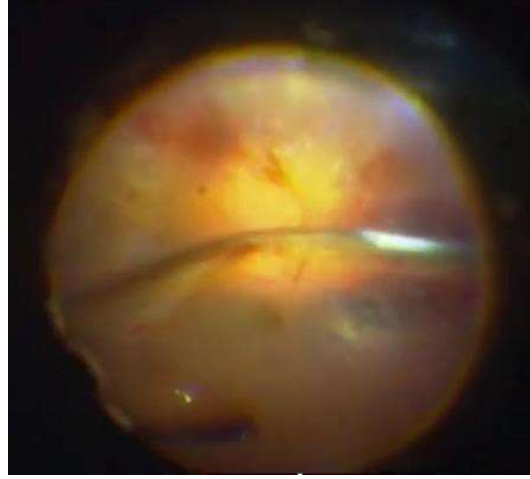
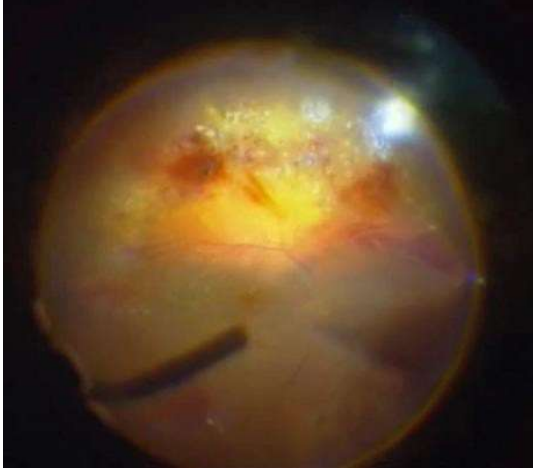
Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

- With the help of the Backflush Instrument on the left, the air in the posterior cavity is removed from the eye. Then the air infusion is switched off and the silicone oil fills the eye cavity.

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- After this process, Backflush Instrument and Silicone Oil Injection Port are removed from the operation area. Then, the operation is completed after the intraocular pressure is adjusted to 8-10 mmHg.

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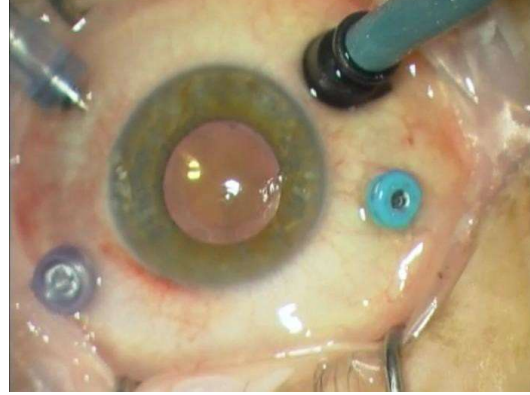
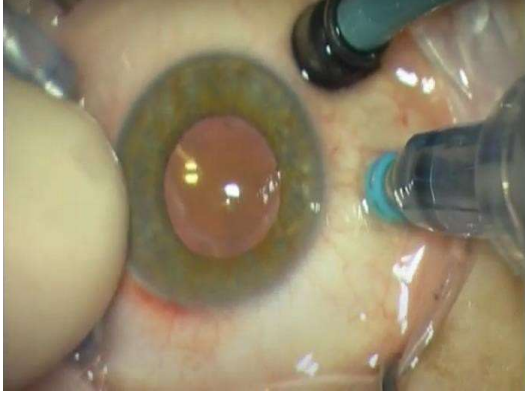
Revision Date
Revizyon Tarihi
20.11.2023

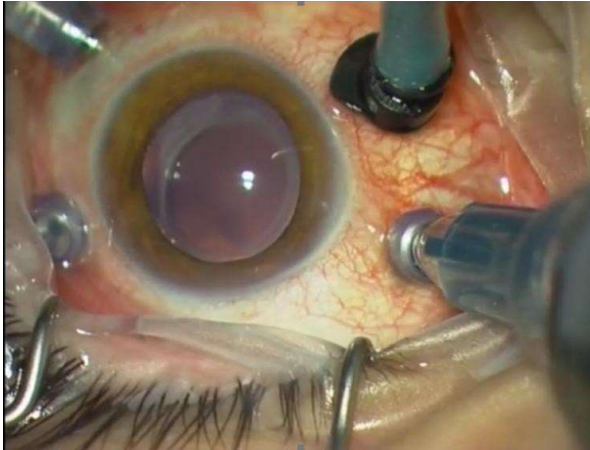
2) Silicone Oil Removal

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- The equipment shown below is placed in the patient's eye to remove silicone oil. Then the inside of the patient's eye is illuminated with light and examined by the surgeon while the procedure is being performed.

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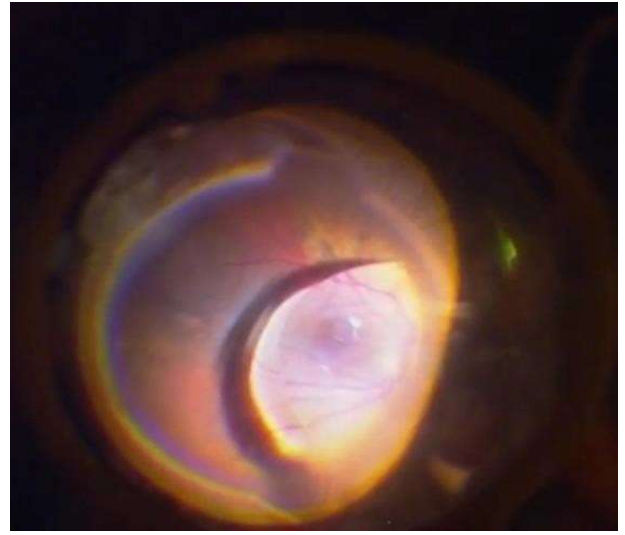
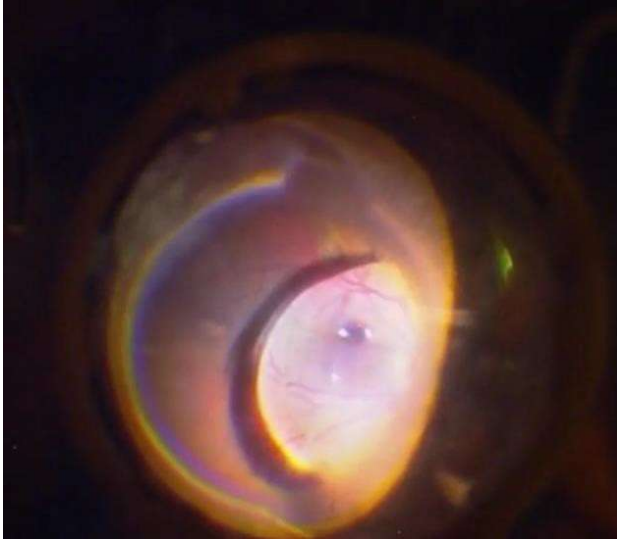
Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

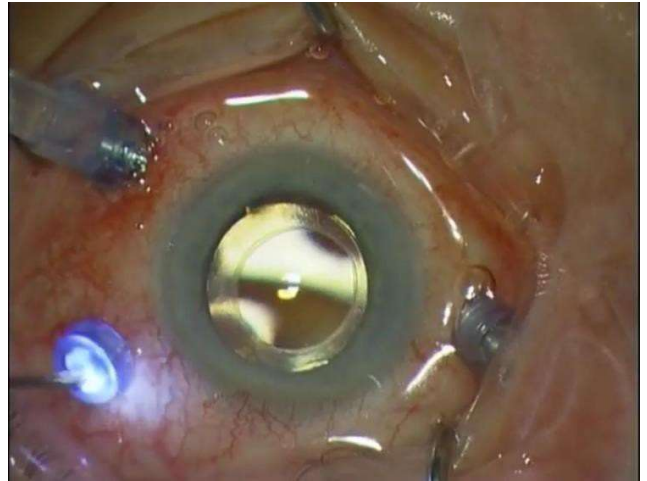
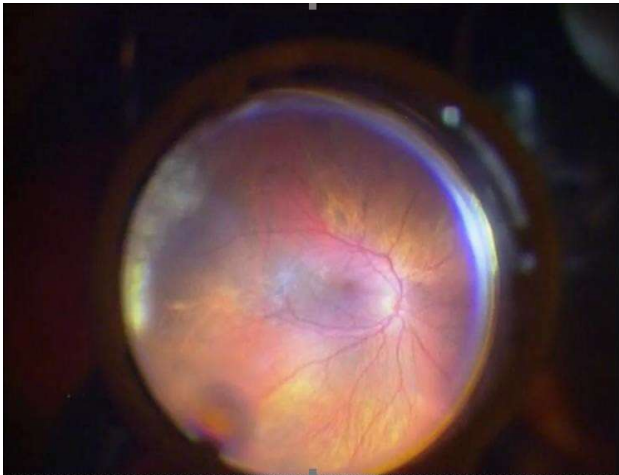
- The silicone oil in the eye is removed with the help of a cannula. At this stage, care should be taken that there are no bubbles in the eye. As seen in the picture below, the bubble can form in the eye, this bubble should be removed as seen in the second image.

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- In this way, all silicon oil remaining in the eye is removed. Later, these tools are sacrificed and the operation is completed.

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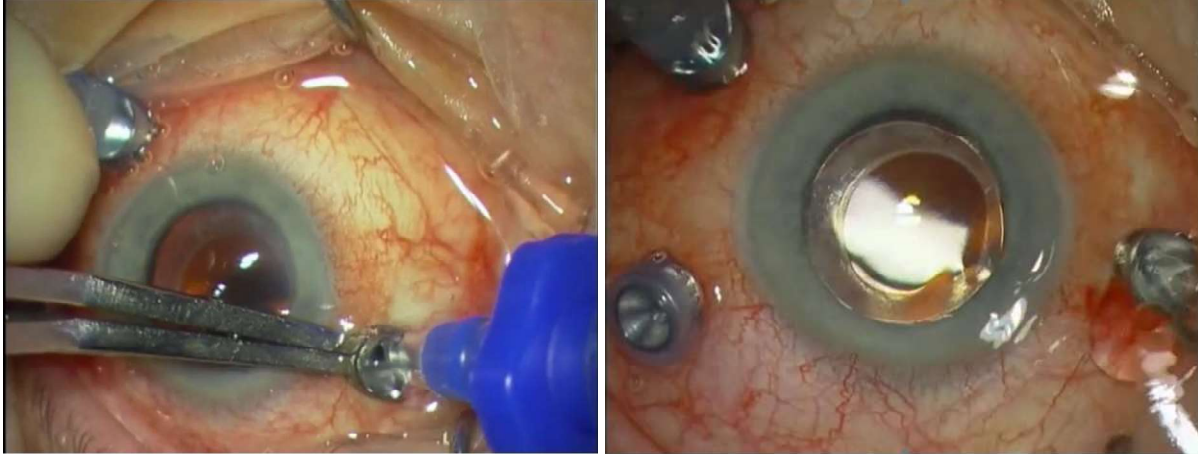
- Apart from this method, the devices indicated in the image below can also be used. These devices are used for the rapid removal of silicon oil. The choice of this depends on the surgeon. In addition, while the operation is carried out with this device, first silicon oil is actively drawn. The device is then removed and the silicone oil is allowed to drain passively. Later, these tools are sacrificed and the operation is completed.

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Factors That Can Affect the Success of the Operation:

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The products are for single use only. There is absolutely no reusability.

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Shelf Life:

Raf Ömrü:

Shelf life of the product is determined as 3 years as a result of stability study.

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Duration of the Product in the Body:

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Silicone oil remains in the eyes of the patient for 3-6 months after the operation is performed, according to the physician's decision. At the end of this period, an additional operation must be performed to remove the silicon oil.

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Product Storage and Shipping Conditions:

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Protect the product against direct sunlight during shipment and storage.

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Keep the product dry in any environment.

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Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Medical Equipment Used with Products:

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Silicone oil products are used with the following medical equipment;

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- o Injector,
Enjektör,
- o Cannulas.
Kanüller.

While Silicone Oil is prepared as a vial, Injector and Cannula are presented to the user with the product. However, while Silicone Oil is prepared as a syringe, Silicone injection adapter product is included in the package content depending on the user's request.

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Accessory:

Aksesuar:

The product does not have any accessories.

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Packaging, Labeling and Handling:

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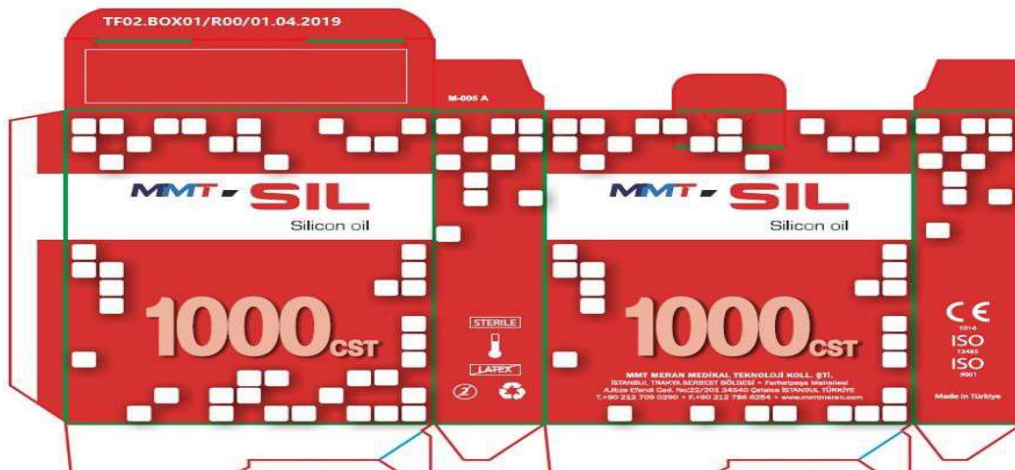
Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, punc-ture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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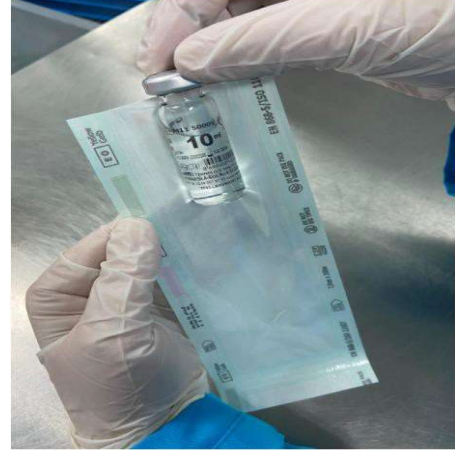
Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

The packaging sample of our products is as seen in the photos below.

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Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

Symbols Prepared According to EN 15223-1 Standard and Their Meanings

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