Document No
Doküman No
TF07-MRN-IFU07-04

Issue Date
Yayın Tarihi
23.09.2020

Revision No
Revizyon No
03

Revision Date
Revizyon Tarihi
20.11.2023

OPERATION GOWN
Ameliyat Önlüğü

Company Name

Firma Adı

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Product Models:

Ürün Modeli

Model and size information of the Operation Gown products produced by us are shown in **TF07.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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Operation Gown is used in all surgical operations to protect hygiene, to ensure safety against infection risk for patients.

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Indications:

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Operation Gown is used in surgical operations in order to prevent patients from infection which may arise from personnel.

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Contraindications:

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Apart from preventing infection, our products are used for radiation etc. It should not be used to protect against such substances.

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Complications:

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If the patient is allergic to the product material, there will be irritation or redness in the areas where the product is in contact with the patient's tissues.

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Side Effects:

Yan etkiler:

There are no known side effects of the product.

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Product Users:

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The product should be used by medical personnel who attend surgical operations.

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Usage Information:

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➤ Before use, check to be sure that there are no defects, holes or tears in the products.

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➤ Do not use it if there are any defects, holes or tears in the products.

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➤ Table Drape with Adhesive Tape is unfolded by holding downward.

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➤ Labels of adhesive tapes are pulled off from the corner of the labels.

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➤ Adhesive tapes are aligned properly on surgical table.

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➤ Instrument Table Drape is laid on instrument table.

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➤ Barrier region of Instrument Table Drape is placed on area where surgical instruments are put.

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➤ Special folded Operation Gowns are held by a person.

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➤ Operation Gowns are unfolded by holding downward.

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➤ The inside of the gown is turned towards the person wearing the gown.

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➤ The person wearing gown wears the gown by extending her arms forward.

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➤ The wrist part of Operation Gown is fitted to the wrist.

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➤ The touch and close fastener on the neck part of the gown is attached.

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➤ Cataract Drape with Pouch is unfolded by holding downward.

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- Pull off the label and adhere the transparent round area on patient's eye on the label located on transparent round area.

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- Put the pouch right downward for right eye and left downward for left eye in accordance with the arrow which indicates direction of the pouch on the label located on transparent round area.

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- Cut the transparent area with a sharp object such as scissors.

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- Medical Towels are taken out and put on a place where surgeon can easily reach.

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- At the end of the surgical procedure, collect all products, place them in the medical waste bag and take them to the medical waste area for disposal.

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Sterility/Sterilization Method:

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The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Shelf Life:

Raf Ömrü:

Shelf life of the product is determined as 4 years as a result of stability study.

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Product Storage and Shipping Conditions

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The ambient temperature where the products are stored must be in the range of +5 / +35 ° C and the humidity at maximum 70% rh.

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The products should be stored in a way not to be exposed to direct sunlight and high above the ground to prevent contact with water.

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Medical Equipment Used with Products:

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The products are used with no additional medical equipment.

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Accessory:

Aksesuar:

The product does not have any accessories.

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Packaging, Labeling and Handling:

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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Revision No
Revizyon No
03

Revision Date
Revizyon Tarihi
20.11.2023

Packaging steps are demonstrated as follows:

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