User Manual

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Product Models:

Ürün Modelleri:

Model and size information of the Ophthalmic Sponges products produced by us are shown in **TF08.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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There are five model of the product. They are “PVA Eye Sponge”, “Absorbent Stick”, “Cotton Swab”, “Lasik Expanded Spear” and “Lasik PVA Spear”.

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PVA Eye Sponge:

PVA Göz Süngeri:

During ophthalmic operations such as cataract surgery, it is used to absorb and remove the discharge from the eye in order for the surgeon to perform easy operation. This product is made of PVA (Polyvinyl Alcohol) sponge raw material.

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Absorbent Stick:

Emici Çubuk:

During ophthalmic operations such as cataract surgery, it is used to absorb and remove the discharge from the eye in order for the surgeon to perform easy operation. This product is made of cotton raw

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Cotton Swab;

Pamuklu Çubuk:

During ophthalmic operations such as cataract surgery, it is used to absorb and remove the discharge from the eye in order for the surgeon to perform easy operation. This product is made of cotton raw material.

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Lasik Expanded Spear;

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The product is used to absorb and remove the currents in the eye from the operation area so that the surgeon can perform easy operation during Lasik surgery applied to the patient. This product is made of cellulose raw material.

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Indications:

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Ophthalmic Sponges are generally used in operations such as cataract and Lasik surgery to absorb the fluids formed in the eye area and to enable the surgeon to perform the operation more easily.

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Patient Population:

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Patients with various eye conditions, especially cataracts.

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Contraindications:

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It should not be used outside of ophthalmic surgery.

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Complications:

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Cross contamination can occur when the same product is used in more than one patient.

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Side Effects:

Yan Etkiler:

There are no known side effects of the product.

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Product Users:

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The product should be used by specialist physicians who have completed the necessary medical training. Users must have operational experience.

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Usage Method:

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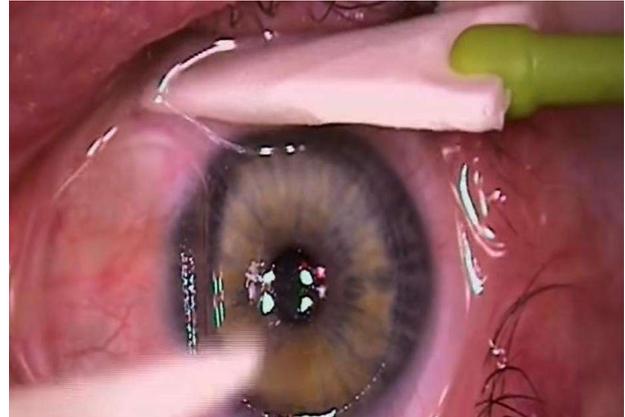
- In eye surgery operations, various fluids are given to the eye area in order not to keep the operation area dry, to make the operation easier and not to damage the eye.

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- The products in the images below are used to absorb unwanted liquids in the operation area.

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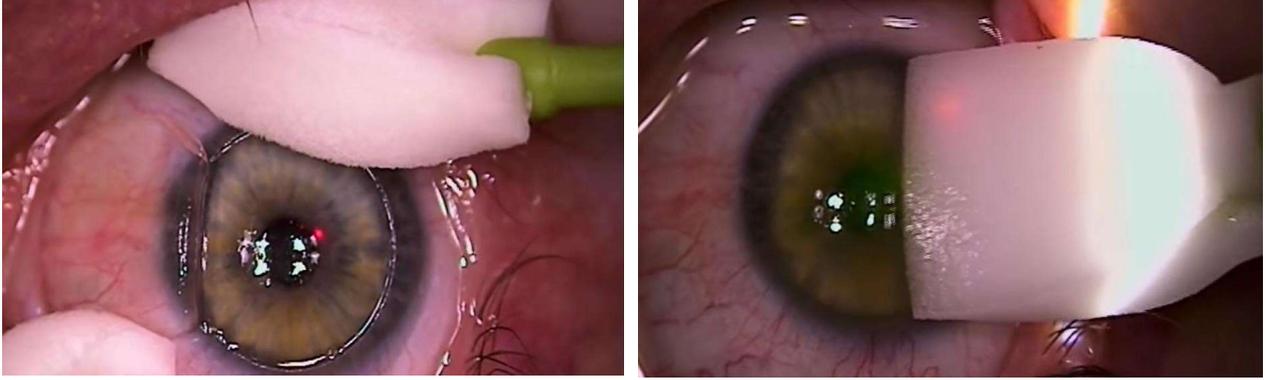
- The products have high liquid absorbency. The products should be brought over liquids in the operation area and kept constant for a while. At the end of this period, the products absorb liquid and swell as shown below.

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Factors That Can Affect the Success of the Operation:

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Sterility/Sterilization Method:

Sterilite/Sterilizasyon Metodu:

The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Shelf Life:

Raf Ömrü:

In line with the stability studies carried out in independent laboratories, the product shelf life was determined as 4 years.

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Product Storage and Shipping Conditions

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Protect the product against direct sunlight during shipment and storage. Keep the product dry in any environment.

Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Medical Equipment Used with Products:

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The products are used with no additional medical equipment.

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Accessory:

Aksesuar:

The product does not have any accessories.

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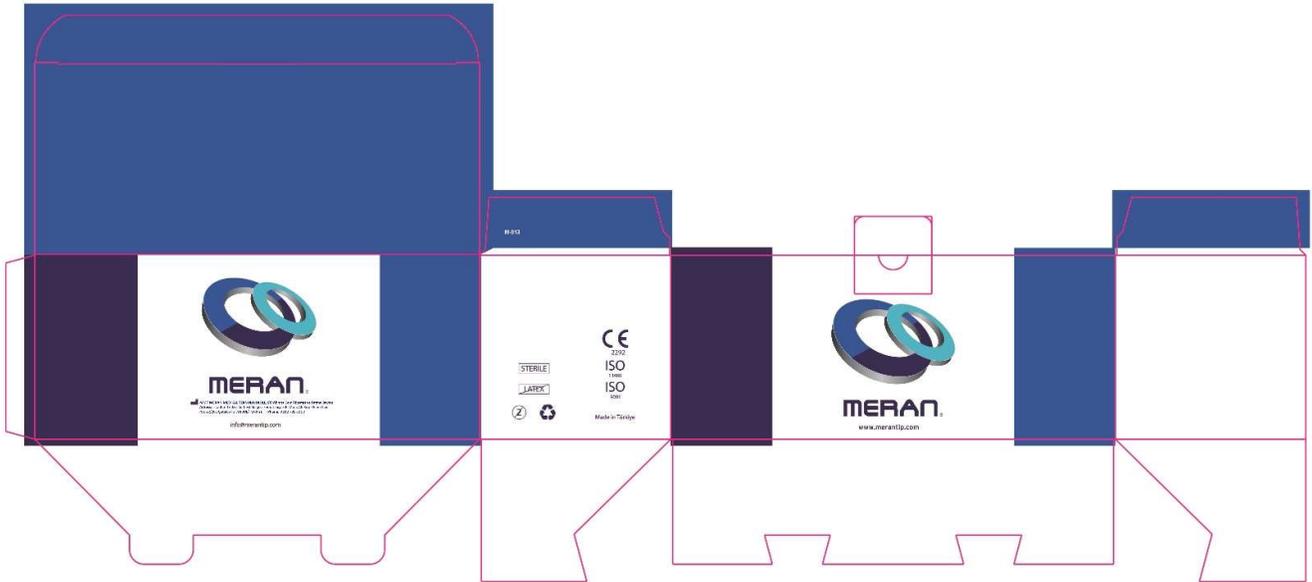
Packaging, Labeling and Handling:

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

The packaging sample of our products is as seen in the photos below.

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