User Manual

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Product Models:

Ürün Modelleri:

Model and size information of the Ophthalmic Speculum products produced by us are shown in **TF08.TB.01 Annex-1 Product Models Table** with pictures.

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Intended Use:

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There are four model of the product. They are “Eye Speculum Adult”, “Pediatric Eye Speculum Small”, “Pediatric Eye Speculum Medium” and “Pediatric Eye Speculum Large”.

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Eye Speculum Adult;

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It is a self-holding ophthalmic surgical instrument used to open the eyelids during the ophthalmological examination or procedure applied in adult patients. The metal speculum produced enables the surgeon to work safely in a wide angle by stretching the eyelids of the patient and stopping involuntary blinking.

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Pediatric Eye Speculum Small;

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It is a self-holding ophthalmic surgical instrument used to open the eyelids during the ophthalmological examination or procedure applied in pediatric patients. The metal speculum produced enables the surgeon to work safely in a wide angle by stretching the eyelids of the patient and stopping involuntary blinking.

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Indications:

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Ophthalmic Speculums are specially used to open the eyelids during the ophthalmological examination or procedure applied in patients. The metal speculum produced enables the surgeon to work safely in a wide angle by stretching the eyelids of the patient and stopping involuntary blinking.

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Patient Population:

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Patients with various eye conditions, especially cataracts.

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Contraindications:

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It should not be used outside of ophthalmic surgery.

It should not be used on infected eyes.

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Complications:

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Eyelid redness may occur.

After operation mild chemosis and redness can occur.

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Side Effects:

Yan Etkiler:

There are no known side effects of the product.

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Product Users:

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The product should be used by specialist physicians who have completed the necessary medical training. Users must have operational experience.

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Usage Method:

- Product model and size are selected according to the patient to be operated.

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- Then the product is placed to hold the patient's lower and upper eyelids, as seen in the image below.

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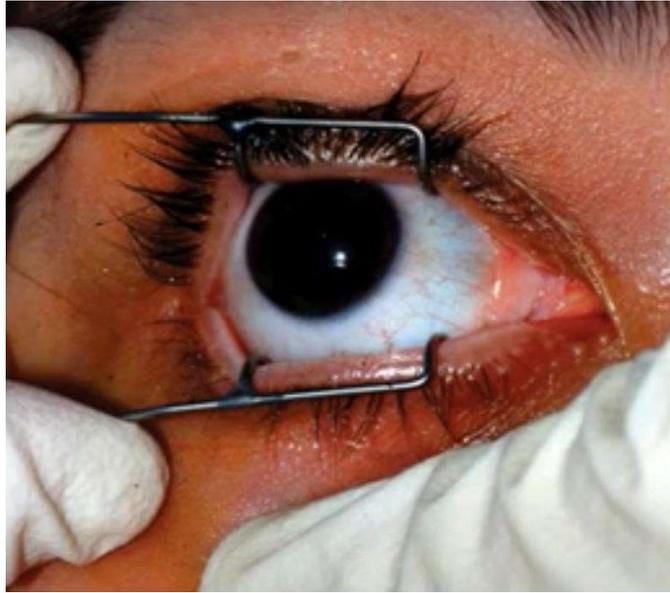
- It is used in a wide variety of eye operations. After the operation is completed, it is removed from the operation area and the operation is terminated.

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Factors That Can Affect the Success of the Operation:

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Sterility/Sterilization Method:

Sterilite/Sterilizasyon Metodu:

The products are sterilized with Ethylene Oxide and presented to the user as sterile.

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The products are for single use only. There is absolutely no reusability.

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Shelf Life:

Raf Ömrü:

In line with the stability studies carried out in independent laboratories, the product shelf life was determined as 4 years.

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Product Storage and Shipping Conditions

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Protect the product against direct sunlight during shipment and storage.

Keep the product dry in any environment.

Store the products between 5 ° C and 35 ° C temperature and maximum 70% RH humidity.

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Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

Medical Equipment Used with Products:

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There is no medical equipment to be used with products.

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Accessory:

Aksesuar:

The product does not have any accessories.

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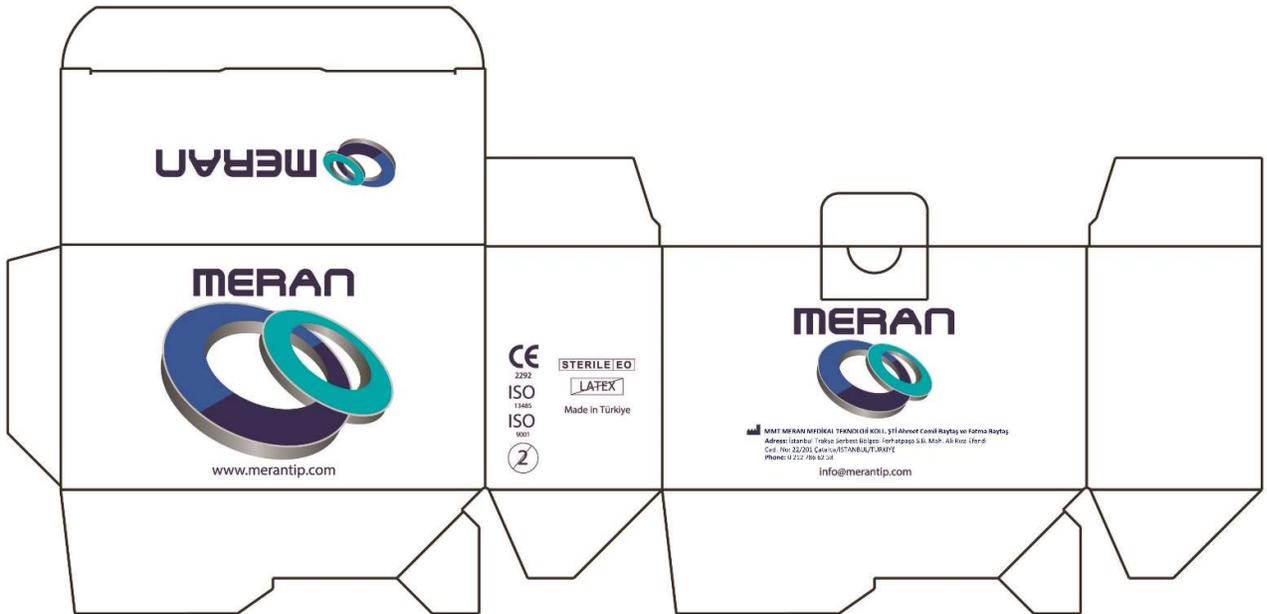
Packaging, Labeling and Handling:

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Packaging material are manufactured according to EN ISO 11607-1 and EN 868-6 standard, tear, puncture and high resistance to liquids, which maintains sterility is the long packaging material. It sealed with heat cut in desired dimensions. Leak-proof barrier allows the product to remain sterile.

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Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

The packaging sample of our products is as seen in the photos below.

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Revision No
Revizyon No
04

Revision Date
Revizyon Tarihi
20.11.2023

Symbols Prepared According to EN 15223-1 Standard and Their Meanings

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